



www.bin-to.co.uk

Deluxe Bed Table Instructions



Instruction for Overbed Tables

The [Bin to](#) over bed range offers a high quality and flexible range of occasional and over bed tables.

1. Carefully unpack the table parts from the box. You should have:

- 2x side frames
- 1x cross brace (with 2x castor wheels)
- 1x table-top (with 2x telescopic tubes)
- 2x threaded handles

2. Assemble the 2 sides frames to the cross brace using the 2 threaded handles (as shown in the photograph below)



3. Slide the 2 telescopic tubes attached to the table-top into the 2 side frames (as shown in the photograph below)



4. Adjust the table top until comfortable, and then screw the two tightening handles clockwise to secure. (Please see the photograph below) To adjust the table top unscrew the handles anti clockwise.



Please Note:

Only unscrew the tightening handles until the telescopic tubes can be adjusted. Unscrewing the handles too much may damage the tightening handle. The handles do not unscrew out of the tubing



Byn-to Ltd
25 Bedgrove
Aylesbury, Buckinghamshire
HP21 7BQ
United Kingdom

T: 0845 602 5565
W: www.bin-to.co.uk

Byn-to Ltd reserves the right to change their specifications without prior notice.