



[www.bin-to.co.uk](http://www.bin-to.co.uk)

## XL Rollator Instructions



- Examine the XL Rollator for shipping damage. Do not use the XL Rollator if there is apparent damage. Please contact your dealer for repair and /or return information.
- Upon removal from box, verify that all screws and bolts are tightly secured.

### Assembly instructions

1. Open XL Rollator frame and insert handles into frame tubing. Adjust to the height that best suits you. Insert triangular knob screw through frame and handle holes. Turn it in a clockwise direction until handles are secure. (See Fig.1)



Fig. 1


2. Place the padded backrest tubing into brackets on the front sides of XL Rollator, making sure the buttons click into brackets holes. (See Fig.2)



Fig. 2


**Warning do not use this product for any other purpose than as a walking aid it is not to be used as a wheelchair of any kind**

3. To use the brakes, pull up the brake handle grips. To lock the brakes, push-down the brake handle grips till they snap into lock position. (See Fig.4)

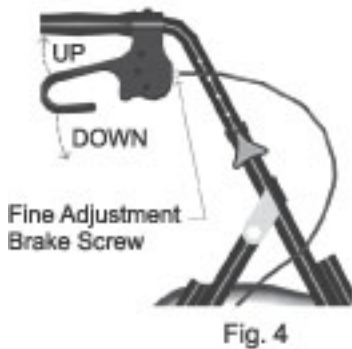

4. To fold the XL Rollator pull out the silver coloured knob located on the right hand side on the cross member, then pull the cross member upwards this will enable you to fold the rollator. When unfolding the Rollator push down on the cross members each side, making sure that the silver locking knob clicks in to lock the Rollator.

(See Fig. 5)

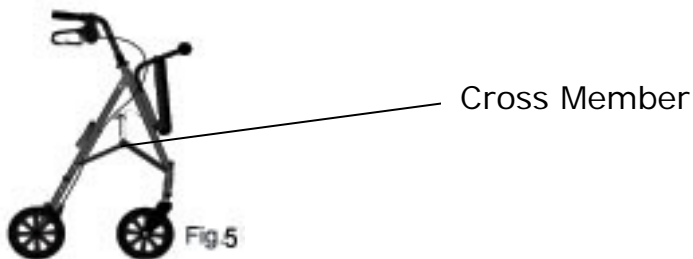

**Warning do not use this product for any other purpose than as a walking aid it is not to be used as a wheelchair of any kind**

**User safety instructions:**

1. Make sure XL Rollator is fully open before use. The side folding bars should be in the locked position (fully horizontal).
2. Ensure that both handle grips are at the same height and tightening knobs are completely secure.
3. Always lock the brakes before attempting to sit on the seat.
4. The XL Rollator is a "walking aid" and should never be used for any other purpose.

**CAUTION: WEIGHT LIMIT FOR ROLLATOR USE: 115KG****Maintenance**

1. Regularly verify that all bolts and screws are properly tightened is highly recommended. Use the "Allen Key Set" provided with the XL Rollator to tighten screws and make adjustments are required.
2. To increase the brake force you have to rotate the fine adjustment brake screw counter-clockwise and the nut clockwise. (see fig.6)
3. Keep the wheel clean.
4. Clean the XL Rollator with soap and water or mild liquid household cleaners. Avoid the use of harsh cleaning solvents, such as acetone, which may attack the plastic parts, painted frame or loosen the handle grip.
5. If a mechanical defect appears, contact your dealer for repair. Never use a defective XL Rollator.
6. Replace wheels if worn or damaged.

**Warning do not use this product for any other purpose than as a walking aid  
it is not to be used as a wheelchair of any kind**

# WARRANTY

Byn-to Limited warrants to the original retail purchaser for those periods set forth below that all new products shall be free from failure caused by defects in material and workmanship when used under normal conditions in a normal manner. The obligation of the company under this warranty consists only of repairing or replacing, at the company's option, any part, which is proved to be defective in material or factory workmanship. The company reserves the right to require the return of defective material to verify the claim. All costs, such as labour, transportation, and loss of use, associated with the replacement of defective parts and components are not covered under this warranty.

## TERMS OF WARRANTY

The product is warranted for a period of twelve (12) months from date of purchase by original user. If used commercially, the warranty is ninety (90) days from date of purchase by the original purchaser. If the product is rented or otherwise used as a demonstrator, the warranty period begins on the date of invoice. Repair parts are warranted for a period of ninety (90) days from date of purchase. Replacement parts used in warranty repairs are warranted for a period of ninety (90) days or the balance of the warranty period, whichever is longer.

## WARRANTY EXCLUSIONS

1. ABS plastic housings and footrest covers (wear items and not warranted).
2. Tyres (wear items and not warranted).
3. Decoration and bulbs (wear items and not warranted).
4. Self repair or replacement parts without permission by the manufacturer.
5. Damaged parts resulting from an accident.
6. Damage caused by improper operation, maintenance or storage.
7. Damage caused by commercial use or use other than normal.

## CONDITIONS WHICH WILL VOID THE WARRANTY

This warranty will be void if the product has been subjected to negligence, abuse, misuse, accidental damage, improper operation, improper maintenance, or improper storage. The warranty is also void if damage is due to unauthorised modifications or alterations.

## DISCLAIMER

THIS WARRANTY IS MADE IN LIEU OF ALL OTHER WARRANTIES, IMPLIED OR EXPRESSED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. Bin-To LIMITED FURTHER DISCLAIMS LIABILITY FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES TO INCLUDE BUT NOT LIMITED TO, PERSONAL INJURY OR PROPERTY DAMAGE ARISING FROM ANY DEFECT IN THE PRODUCT.



**Byn-to Ltd**

28 Newton Road  
Kingskerswell, Newton Abbot  
Devon, TQ12 5AA  
United Kingdom

T: 0845 602 5565

W: [www.bin-to.co.uk](http://www.bin-to.co.uk)

*Byn-to Ltd reserves the right to change their specifications without prior notice.*