



www.bin-to.co.uk

Super Light Transport Chair Instructions





Weight capacity 250 lbs/ 114kg/18 stone

Seat width 18"/46cm

Construction Aluminium

MAINTENANCE TIPS

Proper care and maintenance is important in order to keep your Super Light Transport Chair in good working condition.

Be sure that your chair is in proper working condition at all times.

1. At least once a week, wipe off the frame using a clean cloth. If the chair is exposed to moisture, dry the chair immediately.
2. Clean the upholstery with mild soap solution, rinse and wipe dry. Worn or torn upholstery should be replaced promptly as it may not support your body weight.
3. Inspect the wheels at least once a month. Correct any side play by loosening the lock nut on the axle then tighten the axle just enough to remove side play.
4. Solid rubber tyres require limited upkeep, but need to be cleaned periodically with a damp cloth. Solid tyres need to be replaced if cracked or severely worn.

It is important for the attendant to be familiar with the proper assisting procedures in any situation that requires the use of the Super Light Transport Chair. Safety is the key consideration. Consult your physician, nurse or physical therapist to determine methods most suitable for your individual needs and abilities.

OPERATING TIPS

1. To open the chair, pull up on back support in opposite direction of the footrest.
2. To fold down and store stand behind the chair, pull the release chain towards you while pushing the top back support away from you.
3. To deploy footrest simply pull down the footrest support. Pull the footrest support up when not in use.

SAFETY TIPS

1. Before entering or leaving the chair, engage the dual wheel brake locks against the tyre on both rear wheels. Make sure the front castors are in the forward position before transferring into or out of the chair.
- 2 Do not move forward while leaning forward in the chair.
- 3 Leaning out of the transport chair without proper assistance could cause tipping.

Note: You and your attendant should consult a qualified professional for the correct techniques on ramp negotiations.



Byn-to Ltd
25 Bedgrove
Aylesbury, Buckinghamshire
HP21 7BQ
United Kingdom

T: 0800 8766377
W: www.bin-to.co.uk

Byn-to Ltd reserves the right to change their specifications without prior notice.