




www.bin-to.co.uk

Tri Walker Instructions



Instructions for Use

(Please read all instruction before use)

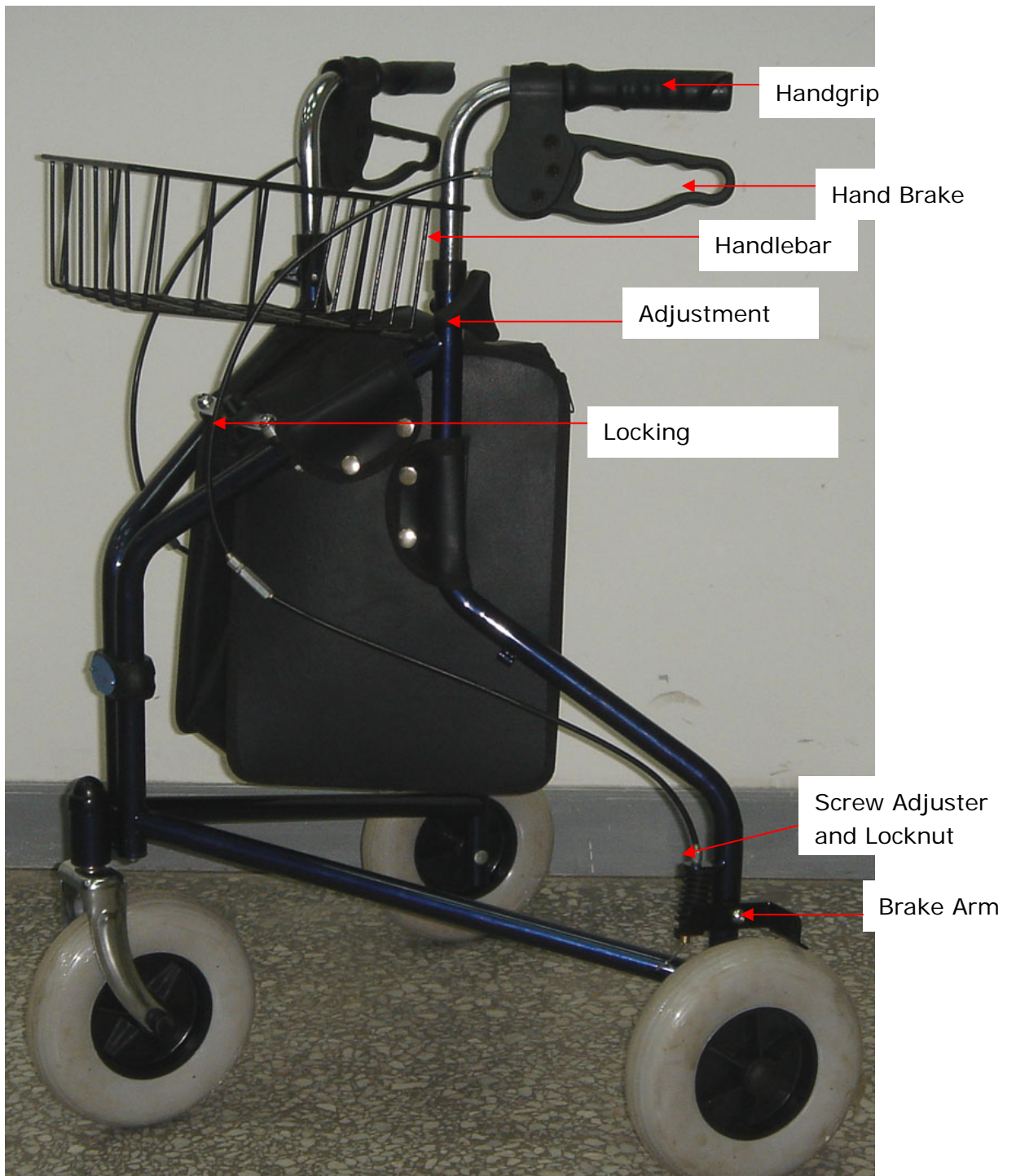

Height Adjustment:

1. Remove Tri Walker from case.
2. Unfold frame: spread legs apart.
3. Push locking mechanism forward (toward front wheel) until it stops.
4. Remove the gray plastic end caps from the frame and discard them.
5. Insert chrome tube sections of left and right handlebars into their respective vertical frame sections.
6. Adjust handlebars to user's height. DO NOT extend the handlebars beyond the knurled (crosshatched) portion of the handlebars. Tighten adjustment knobs on both vertical frame sections until the handlebars are secure.

Operation

Hand Brake: Squeeze hand brake to stop; release hand brake to walk. Ensure that both hand brakes are squeezed at the same time to ensure proper braking. If only one hand brake is squeezed, only one wheel will stop.

To Lock: Push hand brakes down to lock. Ensure that both left and right hand brakes are locked. Always lock hand brakes when user is immobile.

Folding Tri Walker: pull locking mechanism back until it unlocks. Grasp push handles and push inward to collapses Tri Walker.

Maintenance

For dust and dirt, wipe Tri Walker gently with damp, softy cloth.

Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.

Never use organic solvents, abrasive cleaners or scouring pads on any part of (Please read all instruction before use)

Your Tri Walker braking system will periodically require adjustment. Release the locknut and screw out the screw adjuster until the brake arm clears the wheel by approx 3mm retighten the locknut. Repeat the operation on the other side

SAFETY GUIDELINES – please read before use

1. This product has a maximum user weight limitation of 250lbs.
2. Assemble Tri Walker as instructed to ensure safe use.
3. Make sure that all assembled parts are secure and all hardware is tightened before use.
4. If components are damaged or missing, contact your dealer immediately. DO NOT use substitute

parts.

5. Always use caution when using this product on wet or slippery surfaces. DO NOT use Tri Walker on stairs or steep inclines.
6. The Supplier assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
7. Inspect Tri Walker weekly for proper function and wear.

WARRANTY

Byn-to Limited warrants to the original retail purchaser for those periods set forth below that all new products shall be free from failure caused by defects in material and workmanship when used under normal conditions in a normal manner. The obligation of the company under this warranty consists only of repairing or replacing, at the company's option, any part, which is proved to be defective in material or factory workmanship. The company reserves the right to require the return of defective material to verify the claim. All costs, such as labour, transportation, and loss of use, associated with the replacement of defective parts and components are not covered under this warranty.

TERMS OF WARRANTY

The product is warranted for a period of twelve (12) months from date of purchase by original user. If used commercially, the warranty is ninety (90) days from date of purchase by the original purchaser. If the product is rented or otherwise used as a demonstrator, the warranty period begins on the date of invoice. Repair parts are warranted for a period of ninety (90) days from date of purchase. Replacement parts used in warranty repairs are warranted for a period of ninety (90) days or the balance of the warranty period, whichever is longer.

WARRANTY EXCLUSIONS

1. ABS plastic housings and footrest covers (wear items and not warranted).
2. Tyres (wear items and not warranted).
3. Decoration and bulbs (wear items and not warranted).
4. Self repair or replacement parts without permission by the manufacturer.
5. Damaged parts resulting from an accident.
6. Damage caused by improper operation, maintenance or storage.
7. Damage caused by commercial use or use other than normal.

CONDITIONS WHICH WILL VOID THE WARRANTY

This warranty will be void if the product has been subjected to negligence, abuse, misuse, accidental damage, improper operation, improper maintenance, or improper storage. The warranty is also void if damage is due to unauthorised modifications or alterations.

DISCLAIMER

THIS WARRANTY IS MADE IN LIEU OF ALL OTHER WARRANTIES, IMPLIED OR EXPRESSED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. Bin-To LIMITED FURTHER DISCLAIMS LIABILITY FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES TO INCLUDE BUT NOT LIMITED TO, PERSONAL INJURY OR PROPERTY DAMAGE ARISING FROM ANY DEFECT IN THE PRODUCT.



Byn-to Ltd
25 Bedgrove
Aylesbury, Buckinghamshire
HP21 7BQ
United Kingdom

T: 0845 602 5565
W: www.bin-to.co.uk

Byn-to Ltd reserves the right to change their specifications without prior notice.