


instructions to adjust the hand brake system

Important Safety Notice

The hand brake system fitted on this rollator complies with the manufacturer's quality standards. The hand brake system has been preset (adjusted) to the best performance settings. To better suit you personally, please follow the instructions set out below. You may need to adjust the hand brake systems from time to time as they will wear out with normal use.

Hold firmly in place, do not turn.

To tighten the hand brake adjustment: "turn the nut clockwise"

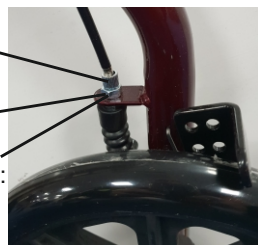

To loosen the hand brake adjustment: "turn the nut counter clockwise"

symbol definitions

The following symbols are found on the Rollator:



Manufacturer



Date of Manufacture



Consult the Instructions for Use



Medical Device

warranty

Your Drive DeVilbiss branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option, supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your local stockist.

maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product.



Drive DeVilbiss Healthcare Ltd, Heathfield Lane,
Birkenshaw, West Yorkshire, BD11 2HW

Tel: 0845 0600 333 | Fax: 0845 0600 334

Email: info@drivedevilbiss.co.uk

www.drivedevilbiss.co.uk

Model:10215 Issue 2019/02

drive

DeVilbiss
HEALTHCARE



go-lite bariatric steel rollator

User Manual

CE

assembly instructions

CAUTION: Do not attempt to adjust or operate Rollator without carefully reading all instructions in this manual.

1. First remove all contents from carton: backrest, handle with tube, Rollator frame and basket.
2. Open frame and slide padded backrest into frame brackets making sure the snap buttons protrude through holes.
3. Unscrew tightening knob from handle and insert handle tube into frame at desired height, lining up holes appropriately. Slide bolts through holes from the inside, fit the fibre washer and screw on the tightening knob, until secure.
4. To fit the basket to the front of the Rollator, locate the hooks on the basket around the frame (either side of the seat) and rest the basket on the lower part of the frame. Alternatively, the basket can be positioned underneath the seat.

Warning! Keep hands away from the folding hinge when folding the Rollator.



safety precautions

1. Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Ensure that both knobs are fully tightened and handles are at equal heights. Test Rollator for stability before applying full weight to hand grips.
3. Test the loop locks before operating the Rollator. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should turn freely.
4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace the wheels if the tyres are significantly worn or damaged.
6. Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

specifications

Dimensions (L x W x H): 69 x 76 x 86.5 - 97 cm

Basket (SWL): 5 kg

Weight: 12 kg

Maximum User Mass: 226 kg (35.5 st)

loop lock operation

Always engage and disengage wheel locks on both sides simultaneously.

1. To Slow Down or Come to a Temporary Stop.
Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.



2. To Temporarily Lock Wheels in Position.
Press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.



Ensure that the brakes are locked on when sitting on the seat.

3. To Release Wheel Locks
Push up on both loop locks with palms of hand to original position.

